


Open for easy cleaning.

Designed to open and close easily for complete cleaning inside shaft, helping to reduce the risk of infection.

- Punch remains in one piece when opened so there are no loose instrument components.
- Forward angle provides precise cutting action.
- Bone ejector is incorporated into the design.
- Made from German stainless steel.
- Quality verified and guaranteed by gSource.

Shown in open position:

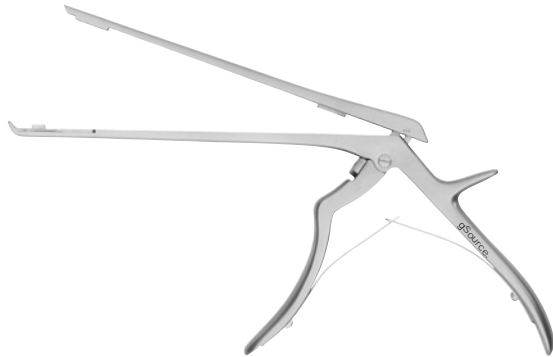


- gS 69.5831** 8" fwd 1mm
- gS 69.5832** 8" fwd 2mm, with ejector
- gS 69.5833** 8" fwd 3mm, with ejector
- gS 69.5834** 8" fwd 4mm, with ejector
- gS 69.5835** 8" fwd 5mm, with ejector
- gS 69.5836** 8" fwd 6mm, with ejector


gPunch EasyClean Spurling Kerrison

Available from your gSource Distributor:

- To open:
- Squeeze and hold both handles together.
 - Rotate knob located in front of hinge screw by 90 degrees.
 - Release handles.

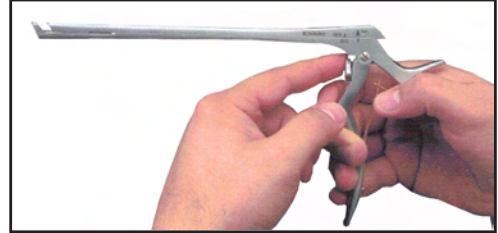

- Pull back slider and lift to open.

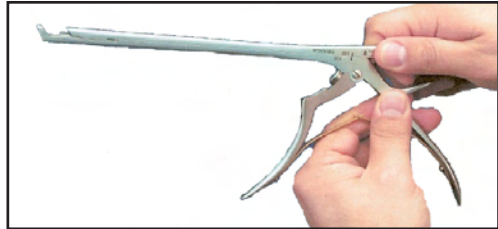

- To close:
- Engage slider and push forward.
 - Squeeze and hold both handles together.
 - Rotate knob to original position.
 - Release handles and check instrument function to ensure slider is engaged properly.

